


Curatron 2000 3-D

The 3-D mattress generates 50,000 microTesla simultaneously over the complete mattress. This guarantees the highest possible penetration through all tissue, cells, and bones deep inside the body. The pulsed electromagnetic field completely envelopes and penetrates the whole body in all 3 axes (length, width and height). This system has been designed for clinical applications where fast results are crucial to both therapist and patient.

How does the 3-D electromagnetic field work?

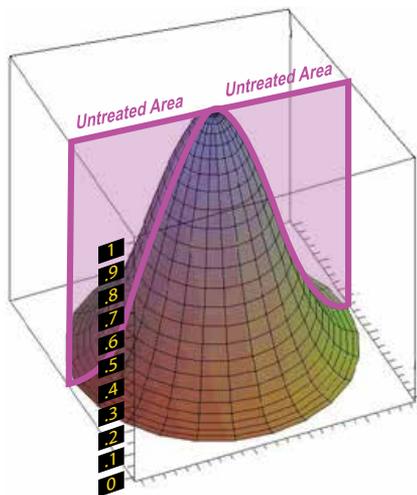

The picture above shows the typical electromagnetic field for one of the flat coils placed inside a therapy mattress. As you move away from the coil the magnetic field diminishes. This means that although the pulsing field will completely penetrate the body, some “dead” areas will occur (as shown in the purple areas of the picture).

The standard Curatron therapy mattress contains 8 large coils inside, which are all activated at the same time. The Curatron mattress has large coils in order to reduce the “dead”


zone effect as much as possible, and to obtain as even a spread of PEMFs as technically possible.

In practice, the “dead” zone effect has no influence on the efficacy of the treatment as people usually do move around somewhat during treatment. Repeated treatments will eliminate this effect almost completely, as nobody lies on the mattress in exactly the same way as in previous treatments.

3-D PEMF Bath

The person is treated with very powerful PEMF's, which can easily be felt with the test magnet, far above the mattress. The whole body is literally “bathed” inside and out by these very strong pulsed electromagnetic fields. This strong PEMF field is still very powerful 20 inches (50 cm) above the complete mattress area.

The body is bathed in a 3-D pulsed electromagnetic field

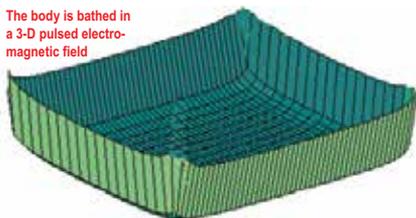

The 3-D ultra-power Curatron system is superior for virtually instantaneous pain relief and treatment of arthritis, osteoporosis, fibromyalgia and other disorders.

A very effective 3-D whole body treatment is assured and patients have no problem relaxing completely on top of the 3-D mattress.



Butterfly Coil for a maximum focused energy field of 160,000 microTesla